

## > ICE-100 Incubation Chamber

### Ideal Operating Temperature-Set

20~40°C

### Dimension and Weight

220\*160\*72mm 1.4kg

### Maximum Operating Altitude

<2000m

### Multiple Slot Capacity

6 Incubation card slots  
Get result in 3 to 30 minutes

### Intelligent Capability

Automatically records the timeout time

### Stability


Instrument life could last 5 years




## > Product Specification

Time Setting	3-30 mins
Temperature Setting	20°-40°C
Humidity	≤95%
Atmosphere Pressure	86.0-106.0KPa
Operating Altitude	<2000m
Power Input	DC12V, 5.0A
Dimension	220x160x72mm
Weight	1.4kg


## > Test Procedure

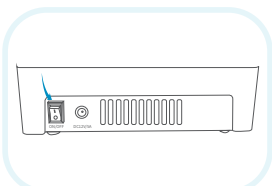


Turn on the power switch

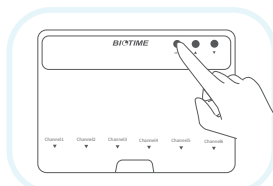

Press the SET button and adjust the temperature setting

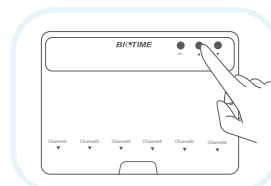

Press the SET button and adjust the time setting

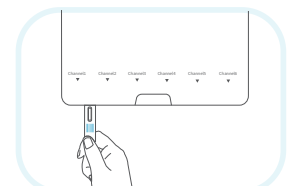

Insert the reagent card and start to incubate